Migo 2G

ROLLATOR – WALKING DEVICE – ROLLATOR

mit Rädern – wheels 190/50 mm
Art.Nr. / REF 723 600 000

CE

Teile des Rollator – Parts of walking device - Onderdelen van de Rollator

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Indication

Walking device – for indoor and outdoor use – for compensation and improvement respectively of restricted ability of walking.

The purpose of walking devices is to compensate for or at least improve restricted walking ability in users (expansion of range and/or mobility).

Persons with restricted hand/arm functions require special adjustments/versions of handgrips or hand rests/arm-rests.

For persons with balance disorders walking frames are not recommended, this is to be checked by an assessment in an individual case. If it is necessary a companion must be present.

Contraindication

Use of the Rollator is not recommended for persons suffering from: perceptual disorders, severe imbalance disorders, an inability to sit.

Scope of delivery contents

1. Walking device
2. Basket
3. Tray
4. Cane holder
5. Operating instructions

Installation

Height adjustment handgrips

Insert the tube of the adjustable handgrip into the frame at least to the "MAX" marking.


Fix it in the desired height with the fixing lever by turning it clockwise. Adjust both handgrips on the same height / level.

Handgrips must be adjusted parallel to the walking direction!

Fixing lever itself can be adjusted in the desired position by simultaneously pulling and twisting.

For readjustment of the height of the handgrips unlock the fixing lever by turning it counterclockwise. Then fix the handgrip how described above.

Unfold

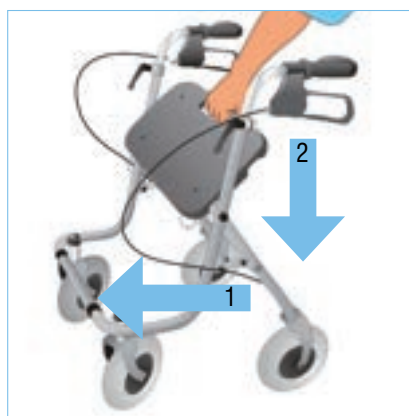Position the walking device on its wheels. Push the front frame forward by pressing on the handgrips down and pulling backwards until the walking device stands on four wheels. Then press the seat down until it clicks into place on the crossbar.

Fitting with basket and tray

Hook in the basket in the hooks at the frame under the seat plate and place the tray with its small feet in the holes in the seat plate (see picture).

Braking / Parking

For braking the walking device pull both brakes upwards to the handgrip (see picture). Always brake slowly and steady.


For parking the walking device press down the brakes until they click in the parking position (see picture).

For unlocking the brakes pull them upwards towards the handgrips until they are unlocked (both brakes are in a middle position).

The effect of the brakes can be affected by abrasion of the wheels. Therefore test regularly the brakes.

Adjustment of brake

For desired braking strength adjust the brake either manually at the screw on the housing of the brake (picture 1) or with the suitable tool (not included) on the brake wire (picture 2).

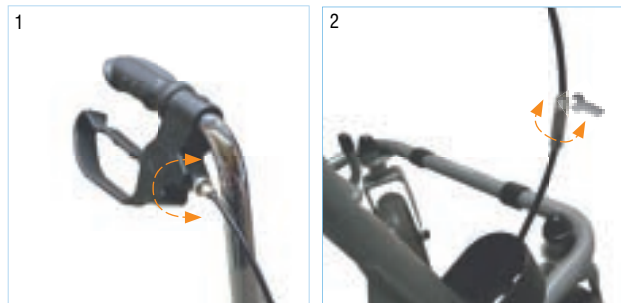Type label

1 Product name / 2 Article number / 3 Serial number


4 Date of manufacturing (Year/Month) / 5 Attention: See operating instructions

Sitting

Position the walking device on an even and solid surface. Then lock the brakes (parking position). Walking device must stand and must not slip. Sit straight up and do not lean backwards (see picture). Seating position only as shown allowed (against the direction)!

For folding pull seat upwards. Walking device can now be transported by trailing or is now suitable for storage (picture C).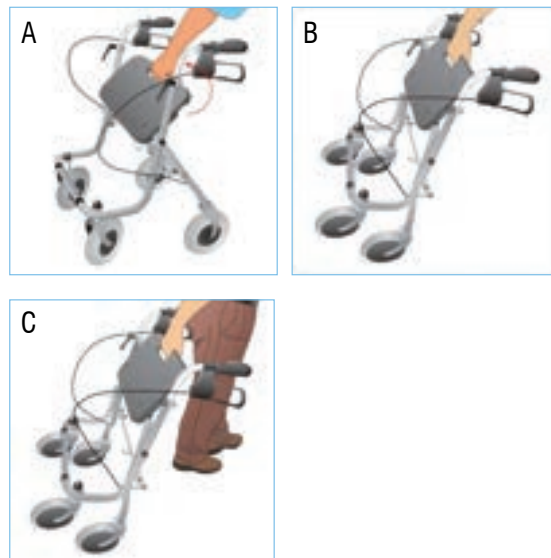

Service of Life

Our company assumes a service life of five years for this product as long as the product is used as intended and all maintenance and service requirements are observed. This service life may be significantly exceeded if the product is carefully handled, repaired, maintained and used and there are no technical limits based on the further development of science and technology.

The service life may also be significantly shortened through extreme and improper use, however.

Determination of the service life by our company does not represent any additional guarantee.

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Drive Medical GmbH & Co. KG • Leutkircher Straße 44 • D-88316 Isny/Allgäu • Germany