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Rollator Standard

Art.-Nr. 11442 010 blau
 Art.-Nr. 11442 020 rot
 Art.-Nr. 11442 030 silber
 Art.-Nr. 11442 035 silber, mit Stockhalter

PZN 03799825
 PZN 03799819
 PZN 03799831

HMV-Nr. 10.50.04.1115

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Please read these notes carefully before use. Be sure to follow the instructions and keep them in a safe place! If this aid is to be used by other persons, they must be fully informed of these safety warnings before use.

A rollator helps those with limited mobility maintain their existing mobility and enables them to enjoy a larger sphere of activities. Staying on the move with a rollator not only serves to improve circulation, coordination and balance, but it also enhances the user's well-being.

This rollator is designed both for use in enclosed areas and in the open.

It is provided with a baggage compartment, a detachable shopping basket and a tray compartment. There is also a seat which can be used for short resting periods. It can be folded easily for transportation, thus saving space.


Safety Indications

- Inspect the product for possible transport damage before first use, paying particular attention to the screw and plastic connections as well as the brake cables and making sure the wheels and wheel suspension are intact. A damaged rollator must not be used.
- Talk to your doctor or specialist retailer to see whether this Rollator is optimally suitable for you and your needs.
- Check that the Rollator is in perfect condition every time before you use it. Check the following components and make sure:
 - all screws, nuts and other parts are firmly tightened
 - all wheel surfaces are intact
 - the height adjustment screws are firmly tightened
 - the brakes function both as parking brakes and hand brakes
 - there are no visible signs of wear on the frames, welded joints and plastic elements.
- Do not use a damaged or defective Rollator!
- Ensure that the Rollator is correctly engaged before every use.
- Always activate the parking brake when you stop or park the Rollator. Always use the hand brakes slowly, evenly and both together.
- The braking effect can be reduced if the tyre surface is worn. Inspect the braking action of the brakes regularly.
- The Rollator should only be used on firm and even surfaces. There is a risk of slipping on dirty, wet, frosty or snow-covered ground! The braking effect is also limited. Please adapt the way you move and walk to these dangers.
- On smooth surfaces such as tiles, stone flooring, PVC, etc. the wheels can block when braking under certain circumstances. Therefore please check the braking performance of your Rollator carefully on these floors and adjust the way you move and walk accordingly. If you wish to sit on the Rollator you should also make sure that the wheels have a sufficient hold on the surface when the brakes are applied and that the Rollator cannot slide away.
- The Rollator should not be used as a support when getting up from a crouching or sitting position! It should only be used for its intended purpose and should not be used for transporting persons or objects over 10 kg.
- Ensure that the handles are adjusted to the correct height for the user in order to avoid improper weight load or falls.
- We recommend you walk between the handles. Always hold both handles; otherwise you can lose your balance.
- Make sure that your hands or clothes do not come into contact with the moving parts of the Rollator! Be particularly careful when folding and unfolding the Rollator.
- The Rollator should be used on the pavement. The Rollator should not be used in moving traffic. For your personal safety on the roads it is recommended that you also wear your own reflectors as well as those fixed to the Rollator.
- Be careful near children. Children should not play with the Rollator!
- The maximum permissible loading for the Rollator is 130 kg. This includes the maximum load in the bag of 10 kg.
- Do not hang any handbags or carrier bags on the handles, the balance of the Rollator can be impaired if you do.
- Do not carry out any adjustments or modifications, which impair the safety of the design.
- Only use accessories approved by the manufacturer.
- Do not carry or pull the Rollator by the brake cables.
- In strong sunlight parts of the rollator, e.g. the handles and seat, can get hot. Please ensure that you cover your rollator up or leave it in a shady place.
- Please prevent skin contact with the Rollator if the Rollator has been in cold temperature for a long period of time in order to prevent freezing.
- Keep the rollator away from naked flames.
- Please prevent running into door frames, furniture, etc. with the Rollator and check the secure position of the Rollator when parked. The frame, connection elements and welding seams may be damaged if the Rollator tips over.
- Please also prevent placing stress on the Rollator when folded together! This could damage the frame, connection elements and welding seams.
- Do not use the Rollator in stairs or escalators.
- Do not lean on the brake lever.
- If you have any queries, please refer to your dealer.

Assembly

Assemble the rollator as followed:

1. Remove all the packaging material.
2. Fold the frame as described under »Unfold/Fold«.
3. Position the rollator so that the steerable front wheels point to the front.
4. Push the handlebars at least 9 cm into the tubing of the walking frame (up to the ridged mark of the handlebar).
5. The screw on the locking lever is fitted from inside towards the outside, through the opening on the coupling clamps. The locking lever is then screwed onto the screw thread. The height of the handles should be adjusted to suit the user. The height can be changed by pushing in and pulling out the handlebars. With the user standing erect the handles should be at the level of the wrists. Make sure that the handles are always aligned parallel to the direction of movement (pic. 2). Pull the locking lever clockwise to tighten it up. In doing so make sure that the groove on the head of the screw fits into the slot on the coupling clamp (pic 1).
6. Now adjust the levers so that the ends point downwards by releasing and turning them. This does not alter the securing effect of the levers!
7. The basket is hung on the hooks at the right and left below the seat.
8. Insert the tray with the little »feet« underside in the holes of the seat.

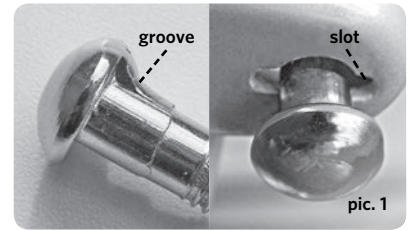Parts:


- | | | |
|--------------------------|---------------------------------|----------------------------|
| 1. Break cable | 6. Baggage compartment | 11. Crossbar |
| 2. Tray | 7. Handle and brake | 12. Back wheel, rigid |
| 3. Seat | 8. Coupling clamp | 13. Front wheel, steerable |
| 4. Fold lock with spring | 9. Security lever | |
| 5. Shopping basket | 10. Break cable adjusting screw | |

Use


Unfold/fold

- When opening out and when folding the rollator, make sure that no clothing or parts of the body become trapped. The rollator can be unfolded by pressing one of the brackets of the back wheels (pic. 3, point 1). Then push the crossbar down (pic. 3, point 2). The fold lock (pic. 3, point 3) automatically folds downwards if the cross strut is in the correct position (pic. 4).
- Check that the fold lock is firmly in position in order to prevent the rollator from folding together in an unwanted manner. When folding the rollator remove the shopping basket and the tray first. Flick the fold lock up (pic. 4, point 2) and pull the crossbar up straight (pic. 4, point 1).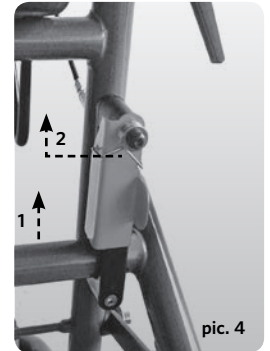


Security lever

- The security levers allow the height of the handles to be adjusted individually. When turned anticlockwise they loosen and the handles can be pushed further in or pulled out (pic. 5, point 1).
- Turn the security levers clockwise and they will tighten (pic. 5, point 2).
- Now adjust the levers so that the ends point downwards by releasing and turning them. This does not alter the securing effect of the levers!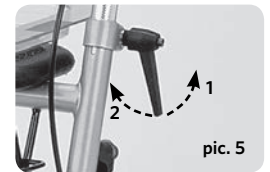

Brake function

- Each of the rear wheels has its own brake lever. Pull both brakes slowly and simultaneously up towards the handles (pic. 6, point 2).
- Securing the brakes: Press the brake handles down until you hear the stop (pic. 6, point 1).
- Unlocking the brakes: Pull both brakes up simultaneously until they are unlocked.
- **Warning:** The brakes can deteriorate as the tyres wear. The braking action should therefore be checked regularly.
- During normal movement the brake block should not touch the tyres. Otherwise this would lead to excessive wear of the surface of the wheels. If you hear grinding noises when moving with the rollator, check the brake settings. The brake cable is fitted with an adjusting screw which can be used to adjust the brakes.

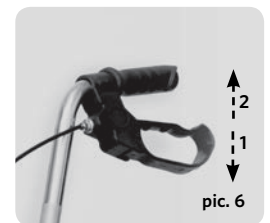Tray

The tray is designed only for small objects. The little »feet« under the tray enable it to be inserted securely in the holes of the seat. Do not place hot, heavy or sharp objects on the tray!

Warning: Do not use the tray as a seat!

Shopping basket

The basket allows the user to transport his shopping conveniently. The basket is hung on the hooks at the right and left below the seat.

Warning: Only use the basket to transport objects and do not overload it (max. 10 kg)!

Seat

In the right seating position (pic. 7) the seat enables the user to take short rests, thus providing relief to tired legs and feet.

Warning: The seat is not a replacement for a chair! Place your weight on it only vertically and do not lean against it slantwise! Make sure that the brakes left and right are in 'parking'-position when you sit on the seat!

Do not lean against the seat! Do not lean over the sides or backwards while sitting on the rollator. That could impair the stability of the Rollator.

Only sit on the seat with your back to the basket. Do not move the rollator when someone is sitting on the seat.

Cleaning/Disinfection

To clean the rollator use water and, if necessary, a mild detergent. Make sure that no water can get into the frame tubes. Thoroughly dry the rollator after cleaning it or when it has come into contact with moisture (e.g. rain). Otherwise there is a danger of rust.

The rollator should only be used on intact skin. A cleaning agent that is gentler to material should be used for manual cleaning. Only DGHM/VAH listed material-friendly surface disinfectant should be used. Please see the relevant manufacturer's instructions for a selection of suitable cleaning agents and disinfectants and their use.

Storage

Store the Rollator light and dust-protected in a dry place. Do not store near a heat source.  

Maintenance

Send your rollator for regular checks to your dealer. He will be able to detect any wear and tear and replace parts, if necessary.

If you find the brake action to be faulty in any way, please contact your dealer immediately.

Re-use

The rollator can be re-used. It must be cleaned and disinfected according to instructions. It must be inspected by specialist shops for any faults and then authorised before it can be used again.

Please order our hygiene and maintenance guidelines.

Disposal

Do not dispose of the rollator in the normal household rubbish. Ask your local recycling authority how to dispose of it properly and in an environmentally friendly way.

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